

ABSORBENT GRANULES

- ✓ MDD Compliant
- ✓ UKCA Marked & CE Marked



AeroHazard™ Absorbent Granules are ideal for solidifying body spills of blood, faeces, urine, and vomit. As the AeroHazard™ absorbent granules are spread over the bodily fluid it produces a gel that makes scooping and disposal quick and efficient. To use this product, simply sprinkle the absorbent granules over the spill area and wait for the gel to form. The gel can take between 1-3 minutes to start working. Remove the gel using a scoop and scraper and place in a biohazard waste bag for safe disposal. Our AeroHazard™ Absorbent Granules come in pots of 240g and must be stored away from children.

AVAILABLE IN:



240g

- ✓ 12 Months
- ✓ Prevents infection & cross contamination
- ✓ Easy to use
- ✓ Safe & Hygienic