



SAM
MEDICAL

SAM® Pelvic Sling II

**SLIDE.
CLICK.
SECURE.**

OPTIMAL FORCE STABILIZATION

SAM® Pelvic Sling is the first and only force-controlled circumferential pelvic belt. Featuring an optimized one-piece design, it's scientifically proven to safely and effectively reduce and stabilize open-book pelvic ring fractures.



FEATURED TECHNOLOGY

AUTOSTOP BUCKLE

The patented SAM® AUTOSTOP buckle acts to avoid over- or under-tightening of SAM® Pelvic Sling. Once optimal compressive force is reached, two prongs activate, clicking to confirm correct application. The sling is secured in place with its hook and loop system strap.



ENGINEERED TO PRESERVE LIFE

1 LOW-FRICTION POSTERIOR SLIDER

Easier transfers following severe trauma made possible by low-friction materials covering backside of the pelvic sling.

2 SAM® AUTOSTOP BUCKLE

Tactile and audible “click” provides clear feedback, confirming proper application and preventing over-tightening while maintaining optimal force.

3 PRECISION STRAP

Precisely-engineered holes lock with AUTOSTOP Buckle, allowing for quick application. Secure fastening with sturdy hook and loop strap.

4 TAPERED DESIGN


Narrow and tapered front facilitates urinary catheterization, interventional radiology, external fixation, and abdominal surgery without removal.

5 RADIOLUCENT MATERIAL

All materials are radiolucent except the buckle's two stainless steel springs, allowing X-rays and CT-scans without removal.

6 3 SIZES AVAILABLE

The Standard size fits 95% of the population. Complete the set with the Small and Large sizes to fit almost any body type.

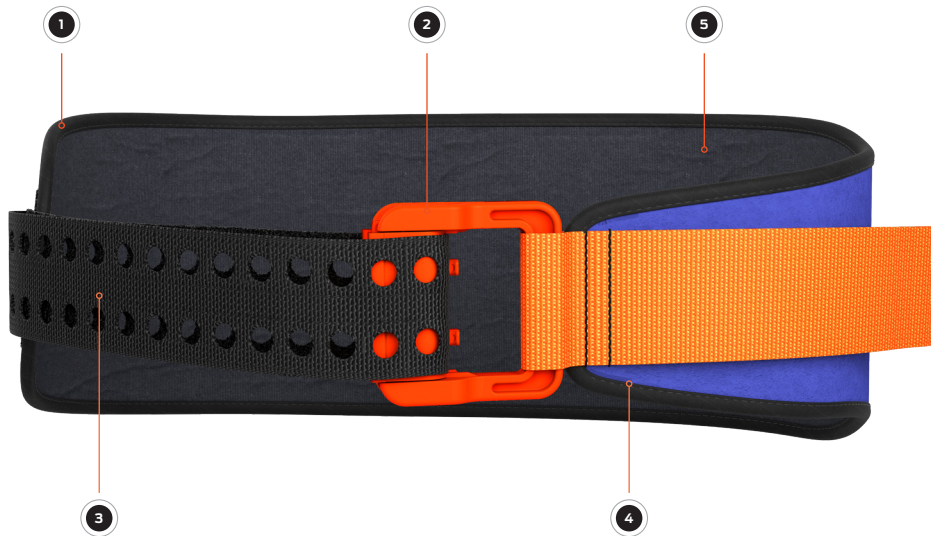

MODELS



CIVILIAN
ORANGE/BLUE

PRODUCT SPECIFICATIONS

PACKAGED SIZE & WEIGHT*

Height: 5.5 in (14 cm)
Width: 7.5 in (19 cm)
Depth: 3.75 in (9.5 cm)
Weight: 8.6 - 9.2 oz (244 - 250g)**

OPERATING PRINCIPLE

Optimal force is reached when 150N of circumferential pressure is attained, which extends prongs from SAM® AUTOSTOP Buckle through the Precision Strap.

MODELS AVAILABLE

Civilian

 Standard 32-50" / 81-127 cm (PS301-OB-EN)



Emergo Europe, Westervoortsedijk 60
6827 AT Arnhem, The Netherlands



SAM Medical Products®
12200 SW Tualatin Rd., Ste 200
Tualatin, OR 97062 USA

* Size and weight vary by configuration and manufacturing process.

**Weight varies by size (Small, Standard, Large)

SAM® is a registered trademark in the United States and other countries.

©2024 SAM Medical Products. All rights reserved.